

Have you seen the full
Bodymedics range?

Visit www.TalarMade.com



TalarMade Ltd, Springwood House, Foxwood Way,
Chesterfield, Derbyshire S41 9RN

Tel: 01246 268456

Fax: 01246 268648

Email: info@talarmade.com

www.TalarMade.com



Usage Instructions

Deluxe Arm Sling

DAS



Usage Instructions

Deluxe Arm Sling

1. Wrap the attached strap around your abdomen with the closed end of the sling closest to the affected arm.
2. Thread the end of the attached strap through the ring on the bottom section of the sling and fasten to itself using the Velcro tab.
3. Take the detached second strap. Thread the smallest end (furthest from the padded section) through the ring above the closed end of the sling (elbow section), fold back and attach to itself using the Velcro tab.
4. Place affected arm into the sling with elbow at the closed end, and the thumb through the thumb loop.
5. Take the padded end of the 2nd strap and wrap around the back of the neck. Thread through the two rings closest to the wrist and secure to itself using the Velcro tabs.

SIZE INFORMATION

For size information, please visit www.TalarMade.com

INDICATIONS

- Immobilisation of forearm, elbow and arm
- Post upper limb trauma
- Post upper limb surgery

WASHING INSTRUCTIONS

- Hand wash in cold water using mild soap
- Do not bleach
- Line dry only



CAUTION

- If irritation occurs, discontinue use and consult your doctor
- Single patient use only