3. The top portion of the device should be in line with your forearm. If not, adjust the Contact Closure Sizing Strap (E) until it is in line with your forearm (Fig. 4).
 4. Pull on the end of each strap until comfortably tight. Do not overtighten the straps. The Web Space should only make contact with the webspace between the thumb and the index finger (Fig. 5).
Note: To loosen the straps, lift on the tab located on each clip until the tension is released (Fig. 6).

Device Removal

To remove the device, pull each Clip toward you to disengage from the Strap Hooks.

Cleaning and care

- Hand-wash using mild detergent and rinse thoroughly.
- Air dry.
- Note:** Do not machine-wash, tumble dry, iron, bleach, or wash with fabric softener.
- Note:** Avoid contact with salt water or chlorinated water. In case of contact, rinse with fresh water and air dry.

DISPOSAL

The device and packaging must be disposed of in accordance with respective local or national environmental regulations.

LIABILITY

Össur does not assume liability for the following:

- Device not maintained as instructed by the instructions for use.
- Device assembled with components from other manufacturers.
- Device used outside of recommended use condition, application, or environment.

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GENERAL SAFETY INSTRUCTIONS

Read these instructions carefully before use. Keep them for future reference. Any serious incident in relation to the device must be reported to the manufacturer and relevant authorities.

The patient should stop using the device and contact a healthcare professional:

- If there is a change or loss in device functionality, or if the device shows signs of damage or wear hindering its normal functions.
- If any pain, skin irritation, or unusual reaction occurs with the use of the device.

The device is for single patient – multiple use.

USAGE

Device Application

1. Unfasten straps and place the hand into the device such that the thumb passes through the Thumb Hole (A) and the Web Space (B) contacts the skin between the thumb and index finger. The Strap Hooks (C) should be on top of the arm (Fig. 2).
2. Fasten straps by placing the square hole in each Clip (D) over the corresponding Strap Hook (Fig. 3).

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